


# SMART INSULIN PENS


## How to use with diasend®

### HOW TO UPLOAD DATA FROM YOUR SMART INSULIN PEN


1. You need a phone with an NFC chip to upload data from a smart insulin pen.
2. Make sure that the pen's dose button is fully depressed, and the dose counter is at zero.
3. Start diasend® Mobile App and log in. The App is available on AppStore (iOS) and Google Play (Android).
4. Hold the smart insulin pen against the phone's NFC chip. The transmission of data starts automatically. When complete you will hear a short ping.

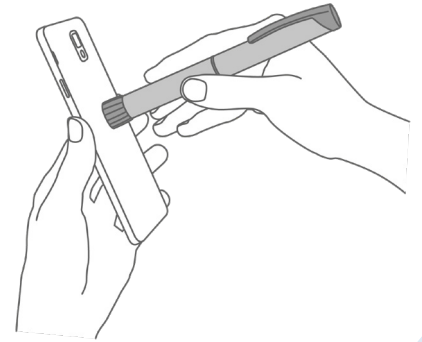


5.

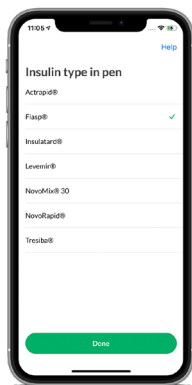

The first time you upload a pen, you will be required to select which insulin is in the pen.

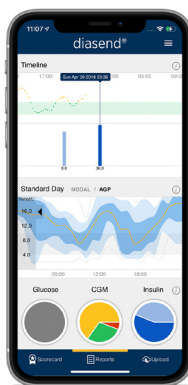

The Reports tab shows your insulin doses based on a timeline.

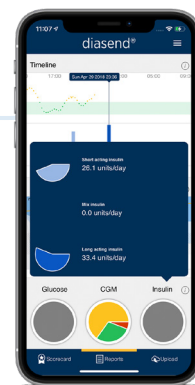

A pie chart of your insulin doses can also be seen in the same tab.

### HOW TO LOCATE THE NFC CHIP ON YOUR PHONE

Normally the NFC chip is fitted at the centre of the back of the phone, but this can vary between different models. Please contact the mobile phone manufacturer if there are any connectivity issues between the pen and the phone and to find out where the NFC chip is located on your phone. Data uploads starts automatically as soon as the phone connects with the pen.