



How to connect your FreeStyle LibreLink app with diasend®

– Synchronize your FreeStyle LibreLink data seamlessly into your diasend® account (*Currently only available outside of US*)



Requirements:


- a diasend® Personal account – **FREE** for patients!
- an Android phone or iPhone and the FreeStyle LibreLink app.



1

Log into your diasend® Personal account at www.diasend.com and synchronize your FreeStyle Libre data to diasend® by clicking the **connect app** button.


Connect app

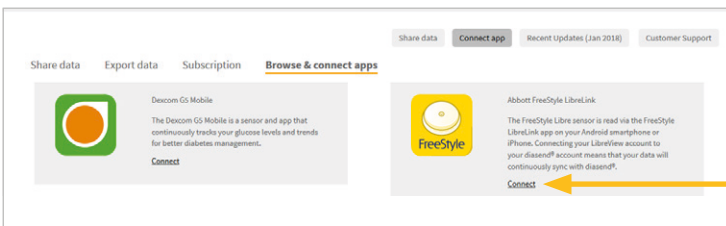

2

Click on the **connect** link for Abbott FreeStyle LibreLink.

Connect

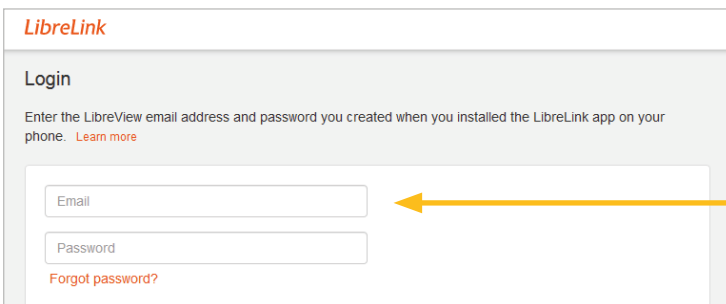

3

Enter your **LibreView login information** to authorize and add the FreeStyle LibreLink app to your diasend® Personal account. Once you have completed steps 1-3, you are ready to upload data.



4

Start the AbbottFreeStyle LibreLink app on your Android phone or iPhone and use it to **scan** your FreeStyle Libre sensor.

The data on your phone will now **appear** in your diasend® Personal account and can be **shared** easily with your clinic. From now on, data will be added every time you scan the sensor with your smartphone.

Please note

If you don't have an Android phone or iPhone you can upload data to diasend® using your FreeStyle Libre reader. You then simply use the diasend® Uploader software on your computer and connect the FreeStyle Libre reader with a micro USB cable to your computer. If you have other devices connected to your diasend® account, FreeStyle Libre data will be shown together with information from the additional devices. FreeStyle LibreLink is a mobile application, developed and provided by Abbott. Use of FreeStyle LibreLink requires registration with LibreView, a service provided by Abbott. The FreeStyle LibreLink app is compatible with NFC-enabled Android phones running OS 5.0 or higher – as well as iPhone 7 or later, running iOS 11 or higher. All trademarks in this flyer are the property of their owners.